

# CARCADDIE™

*Get out of your car with ease!*

- Installs and removes in seconds
- Fits any framed car window
- Nylon construction
- Adjustable length
- Padded handle



## INSTRUCTIONS

Roll your window down enough to loop the end of the CarCaddie™ around the top of the frame of the window. Snap it in place and adjust the strap length such that it is open and you are seated. Move the CarCaddie™ along the frame to find the most effective location for maximum assistance.

## CAUTION

The CarCaddie™ is designed to assist people in standing up and sitting down or to assist carrying items. It is not, however, designed to bear or capable of bearing the full weight of a person. The maximum weight bearing capacity of the CarCaddie™ is 100 pounds. **DO NOT EXCEED THE RATED WEIGHT CAPACITY OR SERIOUS HARM OR INJURY COULD RESULT.** When used as a lift, CarCaddie™ should only be attached to objects that are secure and that can support the maximum rating of the product. Before using, check by pulling on the handle to ensure secure connection.