


Cleaning Instructions for the Ocelco Pillcrusher™

This Metal Pillcrusher is Stainless Steel and easily cleaned.

Here are three options:

- 1) **Commercial stainless steel cleaners:** These are specifically designed for stainless steel and often come in spray or wipe form.
- 2) **Dish soap and water:** Use a mild dish soap diluted in water. Apply with a soft cloth or sponge, rinse thoroughly, and dry to prevent water spots.
- 3) **Baking soda and water:** Make a paste of baking soda and water. Apply to the surface, rub gently, rinse, and dry.

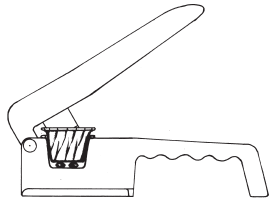

Important Tips:

- 1) Always wipe in the direction of the grain of the stainless steel to avoid scratching.
- 2) Avoid abrasive cleaners or scrubbing pads, as these can damage the finish.
- 3) Rinse thoroughly and dry immediately to prevent water spots and streaks.
- 4) For stubborn stains or fingerprints, try a stainless steel cleaner specifically designed for that purpose.